







DISINFECTION & FOOD SERVICE WIPES

| | Product | Kill/Wet/ Contact Time * | Recommended Surfaces | Category of Disinfectant | Dilution expiration | Lawson # | Notes |
|---|--------------------------------------|---|---|--|---|------------------|--|
|  | Sani-Cloth AF3 (Gray top) | 3 minutes | Hard surfaces - IV poles, BP cuffs, wheel chairs, commodes. Must be used to clean Abbott glucometers. | Quaternary Ammonium. Product is alcohol free | See manufacturer's expiration on container (set at 2 years) | # 99403 | For general disinfection; not for use with isolation precautions. Wear gloves. |
|  | Sani-Cloth Bleach (Orange top) | 4 minutes | Compatible with hard surfaces ** Specific equipment like CareFusion Alaris pumps. | Sodium hypochlorite (Bleach) | See manufacturer's expiration on container (set at 11 month expiration) | # 99404 | For all isolations, including <i>C.diff</i> , <i>Norovirus</i> . Wear gloves. |
|  | Sani-wipe (Red top) | 1 minute | Food service items -dining tables, coffee pots, refrigerators, microwaves, office desks/mouse/keyboard, toys. | Quaternary Ammonium. Product is alcohol free | See manufacturer's expiration on container | # 592232 | Gloves not needed. |
|  | PeriSept | 2 minutes (Hepatitis B/C in 5 minutes) | Not recommended for copper, brass, granite, marble, or zinc. Nor used on unsealed/uncoated marble or terrazzo floors. | Peroxyacetic acid / hydrogen peroxide | 24 hours | Used only by EVS | General use disinfectant for EVS. |
|  | NorOxyCdiff | 2 minutes (Hepatitis B/C in 5 minutes) | Not recommended for copper, brass, granite, marble, or zinc. Nor used on unsealed/uncoated marble or terrazzo floors. | Peroxyacetic acid / hydrogen peroxide | Test before use | Used only by EVS | Used by EVS for 'misting' |

*Surface must stay wet for the specified time. Do not wipe dry.

**To potentially decrease risk of cracking vinyl, wipe surfaces after kill time with water and a clean cloth. Wear gloves.

DISINFECTION & FOOD SERVICE WIPES

| | Product | Kill/Wet/ Contact Time * | Recommended Surfaces | Category of Disinfectant | Dilution expiration | Lawson # | Notes |
|---|---------------------|-----------------------------|---|-----------------------------|------------------------|-------------|---|
|  | Virex II 256 | 10 minutes | Compatible with most medical device material and hard surfaces (Check with manufacturer of equipment if not sure) | Quaternary ammonium | 24 hrs | Used by EVS | Used by EVS for disinfecting surfaces in patients rooms |
| | | | | | | | |
| | | | | | | | |
| | | | | | | | |
| | | | | | | | |

*Surface must stay wet for the specified time. Do not wipe dry.

**To potentially decrease risk of cracking vinyl, wipe surfaces after kill time with water and a clean cloth. Wear gloves.