


PUREWICK®

FEMALE EXTERNAL CATHETER

Do you have a patient who is a good candidate?

- ✓ Patient requiring urine output monitoring but does not meet indications for indwelling Foley catheterization
- ✓ Urinary incontinence and/or frequent urination
- ✓ Difficulty walking from bed or chair to toilet
- ✓ Difficulty using a bedpan
- ✓ Post-surgical or procedure immobility
- ✓ Skin injury or irritation related to urinary incontinence or diapers
- ✓ Bedrest orders

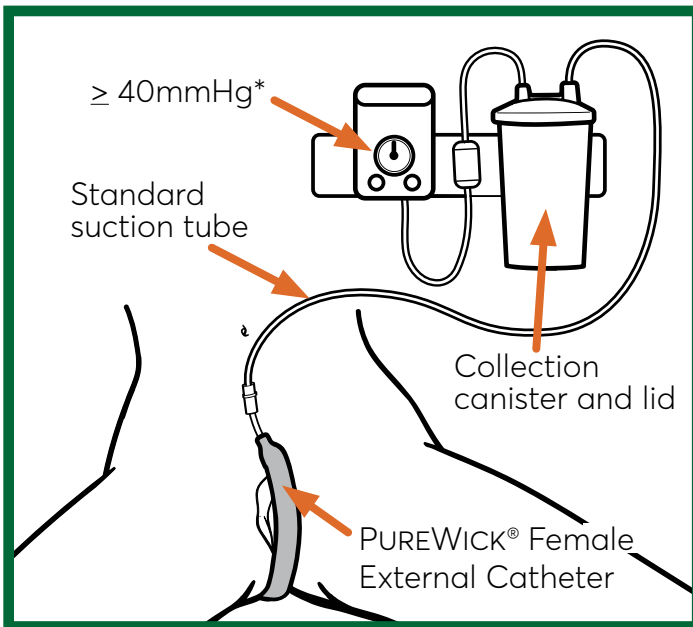

*Always use the minimum amount of suction necessary.

Tips and reminders for placement and removal†

- Replace and perform perineal care every 8-12hrs or when soiled.
- Completely separate legs, gluteus muscles, and labia during placement and removal. Snugly position between labia with the bottom end tucked between the glutes.
- The anatomy holds the product in place for most patients. Mesh underwear may help secure the product with very thin or frail patients.



†Please consult package insert for additional details



To learn more about the PureWick female external catheter, scan this code for the in-service video.


BAIRD
has joined BD



PUREWICK®

FEMALE EXTERNAL CATHETER

How does the PUREWICK® female external catheter work?

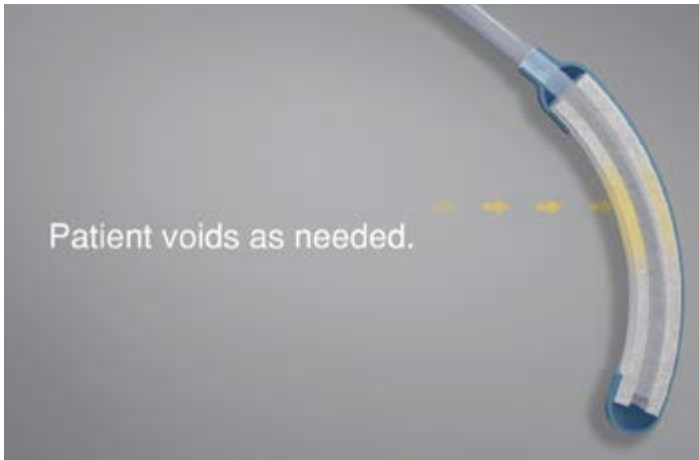

After proper placement of the PUREWICK® female external catheter, patients are able to void as needed.



The PUREWICK® female external catheter absorbs the urine and pulls it down into the base of the device.



The low pressure suction pulls urine away from the base of the device.



Urine is removed from the device and collected in the designated suction canister.

Indications: The PUREWICK® Female External Catheter is intended for non-invasive urine output management in female patients.

Caution: This Product Contains Dry Natural Rubber

Contraindications: The PUREWICK® Female External Catheter should not be used on patients with urinary retention.

C. R. Bard, Inc.
Covington, GA 30014
800.526.4455
www.bardmedical.com

Bard, Advancing Lives and the Delivery of Healthcare and PureWick are trademarks and/or registered trademarks of C. R. Bard, Inc.
©2017 C. R. Bard, Inc. All Rights Reserved. 1712-06 BMD/FECs/0218/0047

Warnings: To avoid potential skin injury, never push or pull the PUREWICK® Female External Catheter against the skin during placement or removal. Never insert the PUREWICK® Female External Catheter into the vagina, anal canal, or other body cavities. Do not use the PUREWICK® Female External Catheter with a bedpan or any material that does not allow for sufficient airflow. Discontinue use if an allergic reaction occurs. After use, this product may be a potential biohazard. Dispose of in accordance with applicable local, state and federal laws and regulations.

Please consult package insert for more detailed safety information and instructions for use.

BARDD
has joined **BD**

