



VIAL2BAG ADVANCE D™

20mm Admixture Device In-Service Introduction

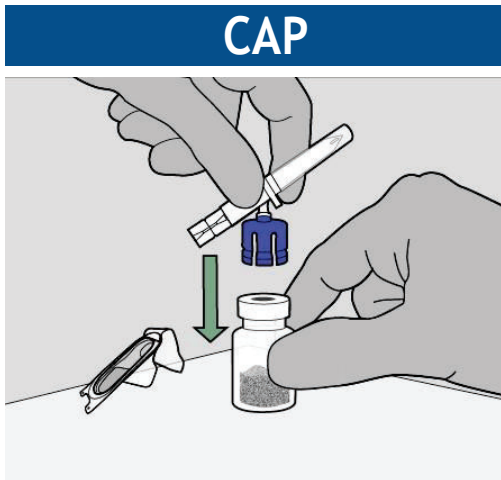


Hold on flat surface

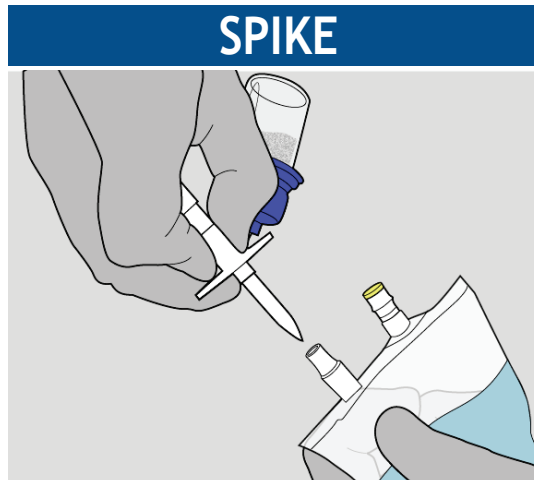

Twist spike while inserting



Scan or
click to
view a short
instructional
video

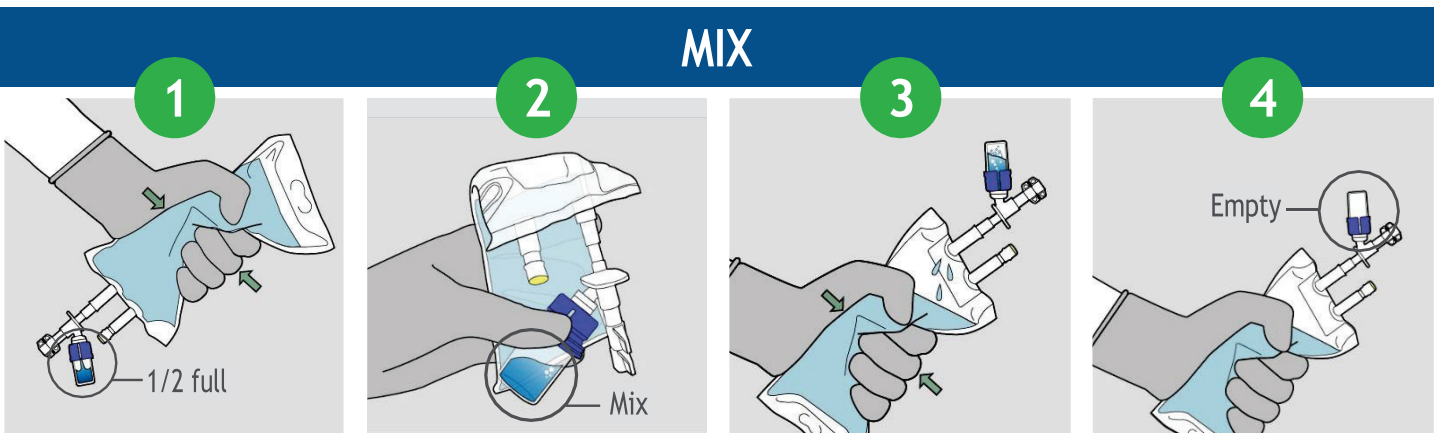

Dissolve or dilute drug before transferring & Repeat 1-4 as needed

Consult Vial2Bag Advanced™ 20mm Admixture Device IFU for full product preparation and use detail



Vial2Bag Advanced™
20mm Admixture Device
Item # 36098150

Quantity
50/Box
150/Carton



Distributed by: Progressive Medical, Inc. 997 Horan Dr. • Fenton, MO 63026
Ordering Information: Phone: 314-961-5786 • Fax: 314-961-4535
cs@progressivemedinc.com