

**Novant Health Disinfection Grid**

Product	Kill/Wet/Contact Time	Recommended Surfaces	Category of Disinfectant	Lawson #	Notes
Gray Top Sani-Cloth AF3  (Novant standard)	Wipe (wet) all surfaces and wait 3 minutes prior to use. Must stay wet for 3 minutes; Reapply as needed. <b>DO NOT WIPE DRY</b> before 3 minutes has elapsed.	Compatible with most medical device material and hard surfaces (Check with manufacturer of equipment if not sure)	Quaternary ammonium, product is alcohol free	Two Size Wipes # 99403 6"X6.75"  #481883 7.5"x15" (for stretchers and other large equipment)	Must be used to clean Abbott glucometers Not for use with isolation precautions Always wear gloves when using this product. Sani-clothes are stable at any temperature. Lawson #481883 7.5"x15" (for stretchers and other large equipment)
Orange Top Sani-Cloth Bleach (Novant Standard)	Wipe (wet) all surfaces and wait 4 minutes prior to use. Must stay wet for 4 minutes; Reapply as needed <b>DO NOT WIPE DRY</b> before 4 minutes has elapsed.	Compatible with hard surfaces	Sodium hypochlorite (Bleach)	# 99404	Use for <b>ALL</b> isolation precautions Always wear gloves when using this product.* PDI suggest to potentially decrease risk of cracking vinyl-wipe surfaces after disinfection/kill time wipe with water and a clean cloth.  May be used in place of Glucochlor clothes to clean telemetry boxes and equipment. May <b>NOT</b> be used on computer monitors or Abbott glucometers.
Clorox Healthcare Germicidal Bleach and Dispatch	5 minutes (Spray and must stay wet for 5 minutes, reapply as needed)	Compatible with hard surfaces	Sodium hypochlorite (Bleach)	#112754	<b>Please note: May cause irritation to eyes &amp; mucous membranes. Please use PPE as indicated. Used for C-Diff and other organisms requiring Contact Enteric isolation</b>
Virex TB	10 minutes (Spray and must stay wet for 10 minutes, reapply as needed)	Compatible with most medical device material and hard surfaces (Check with manufacture of equipment if not sure)	Quaternary ammonium	#381800	<b>Use for Blood Contamination /Blood Spills</b>
Food Grade Sanitizer: Multi-Quat Sanitizer Oasis 146	1 minute for Toys	Reusable Toys (PEDS ED, PEDs etc), Ice Machines, food prep surfaces	Quaternary ammonium	Food Services only	Disinfect surface with sanitizer, let air dry. For toys: Rinse with tap water after air drying.
Disinfectant Clorox Hydro Spray	30 seconds for fabric  1 minute for hard services (Spray and must stay wet for 30 seconds for fabrics / 1 minute for hard surfaces, reapply as needed)	Fabric  Hard Surfaces (can be used on Isolettes)	Hydrogen Peroxide (contains alcohol)	#454782	<b>For fabric:</b> Use zig zag motion spray mist across fabric until wet 30 seconds. Do not saturate. Allow to dry.
Virex Plus (EVS)	5 minutes (Apply and must stay wet for 5 minutes, reapply as needed)	Compatible with most medical device material and hard surfaces (Check with manufacture of equipment if not sure)	Quaternary ammonium	Used by EVS	Used by EVS for disinfecting surfaces in patients rooms